LucRo
by schein

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Directions for use

For patients and orthopaedic staff

Dear Customers,

Thank you for your confidence in us. You have made a good decision with your purchase. It is our goal and objective in close cooperation with our partners, doctors, health insurance companies and medical services to guarantee the best possible treatment for your feet. Learn more about our products and visit our website:

www.lucro.de

Before using our **medical product**, please read carefully through the directions for use and note the following instructions, particularly the safety information:

1. Intended purpose/Indication:

The special shoes are suitable for persons with diabetes mellitus

- with pronounced neuropathy (loss of sensation),
- with pronounced angiopathy (circulatory disorders),
- with healed ulceration and/or other mechanical irritations leading to regional pressure peaks, or
- for minimizing the risk of threatened dorsal ulceration.

2. Place of use/conditions for use:

- The street shoes are for outdoor use only and the indoor shoes for indoor use only.
- The models for outdoor use are intended to be worn on the street – please dry these after wearing in the rain (according to the care instructions on p.3).
- The LucRo slippers are only intended for indoor use.

Please note that there is a danger of slipping on damp or smooth surfaces, or on snow and ice.

Special care should be taken here.

3. Directions for use:

- Always use a shoehorn to put on the shoes.
- Open and loosen the laces or fastenings before putting on the shoes. Put the shoes on and then fasten them.
- In order to avoid pressure marks and circulatory problems do not fasten or lace the shoes too tightly. Please check the fastening right away and adjust it if necessary. If the fastening is too loose, the foot may slip inside the shoe and this should be avoided.
- The shoes constitute a medical product, which is always only intended for one particular patient. The shoes should not be used by another person.
- Your doctor will decide how long and how frequently the shoes should be worn.
- We recommend changing shoes after one day's wear.

Please read our instructions before wearing "LucRo shoes", especially before wearing them for the first time

4. Safety guidelines:

- Feel inside the shoes each time before you wear them and remove any foreign objects.
- Never wear the shoes without LucRo insoles or special insoles from your orthopaedic shoe supplier.
- Only wear the LucRo shoes with socks. We recommend special socks for diabetics.
- New shoes should only be worn for a short while to begin with and your feet checked for pressure marks after 20 minutes. The time and duration of daily use will be decided by your doctor.
- If any pressure marks, pain or other negative occurrences arise, please consult your doctor or orthopaedic shoe supplier immediately.
- Shoes that are too loose can lead to tripping or pressure marks through slipping inside the shoe.
- You should check your feet daily for reddening or pressure marks.

Important note:

If there are any problems with your feet or the shoes, please contact your doctor or orthopaedic shoe supplier immediately!

5. Risks and contraindications of use:

- If you suffer from any skin allergies towards the materials used, please consult your doctor before use.
- Only wear the shoes once any previous ulceration has healed completely.
- Ready-made LucRo shoes are not suitable for significantly deformed feet.

6. Material specifications:

Various types of **smooth** or **suede leather** and **textiles** are used for LucRo shoes.

- **Shoe uppers:** shaft material – smooth- and suede leather, as well as textiles (The textiles used depend on the model – fleece, mesh materials, canvas, microfibre and felt are used)
- **Lining material:** Microfibre
- **ootbed:** Poly-urethane
- **Soles:** Poly-urethane, ethylene-vinyl acetate

The materials for the lining and footbed are the same in all models.

7. Cleaning and care instructions:

Before the first wear

- You should feel inside the shoes each time before you wear them and remove any foreign objects.
- Spray the upper material of the shoes with a standard water-repellent spray before wearing them for the first time. This protects them from moisture.
- Repeat this procedure after you have worn them several times.

Cleaning

- Brushing is often sufficient to clean soiled areas. For this, we recommend a stiff brush for smooth leather and textile uppers.
- A soft-bristled brush should be used for all suede leathers and microfibre.

Disinfection

- The lining material can be cleaned with a damp cloth if necessary. We recommend using a suitable disinfectant to clean the lining of shoes for diabetics (this can be obtained from a chemist, for example).
- Do not use disinfectant on the leather uppers. Please do not wash the special LucRo shoes in the washing machine.

Leather care

- Isolated scratches or insect bite scars in the leather are a sign of its genuineness. To keep the leather supple, the shoes should be treated regularly with nutrients.
- Different types of leather should be treated with the respectively suitable shoe care products.

Drying

- Never put wet or damp shoes directly next to a heat source.
- Please avoid radiators and direct sun for drying the shoes.
- Stuff wet or damp shoes with newspaper or use a suitable shoe tree.
- Give your shoes time to dry. This can take a day or two depending on how much moisture the shoes have absorbed.

Storage

- Please store the shoes in a dry place at room temperature.

You can find detailed tips for cleaning and care on our homepage at www.lucro.de/pflegertipps/

8. Disposal:

- The disposal of the product and packaging should comply with the legal regulations.

9. General tips / maintenance:

- Various factors can affect the service life/durability of the shoes.
- The service life and durability of the shoes is three years with intended use and normal care.
- Please note our care information.
- Check the shoes regularly for functionality, wear and tear or damage with regard to the fastenings, sole etc.
- If there is a loss of function or excess wear and tear or damage, you should discontinue wear of the shoes and consult your doctor or orthopaedic shoe supplier.
- The shoes must be replaced by new ones. Arrange regular appointments for checking your shoes and enter these in the chart at the end of the directions for use.
- We likewise recommend that you have your feet checked at regular intervals (see recommendation below, fig 1).

10. Directions for assembly:

- Remove the filling material (e.g. paper) from the shoes and try on the shoes.
- Adjust the upper material as required (e.g. correct the width, strap length).
- Fitting a sole stiffening is **ONLY** necessary for LucRo

ergonomic models – the stiffening must be glued in place for diabetics; for persons with rheumatism the use of the supplied sole stiffening is decided according to symptoms. It is very important to read and follow the fitting instructions.

- With individual alterations to the product, we point out that the accessory must be permanently marked as "custom-made".

11. Guarantee:

We guarantee the statutory warranty when the shoes are worn as intended.

We recommend having your feet checked at regular intervals. The "National Care Guidelines" list can serve as orientation here.

Class	Findings	Examination	Risk
0	No sensory neuropathy	Yearly	Low
1	Sensory neuropathy	Every six months	Increased
2	Sensory neuropathy and signs of arterial occlusive disease and/or foot deformity	Every three months	Increased
3	Previous ulceration	Every 1 – 2 months	High

Figure 1: National Care Guidelines for Type 2 Diabetes: Prevention and treatment strategies for foot complications

Please inform the schein orthopädie service KG firm of any issues, problems or shoe deficiencies.

We hope you will enjoy wearing our shoes.

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